


# Mepilex<sup>®</sup> Border Flex

Flex Technology for improved stay-on-ability and conformability<sup>1,2,3</sup>


- Easy-to-use five-layer foam dressing for many wound types.
- Improved exudate management - normal and viscous exudate<sup>4,5</sup>.
- Bacteria trapping properties<sup>6</sup>.
- Exudate Progress Monitor - track and record fluid.

## How to use Exudate Progress Monitor

\* All images are stylised for ease of visualisation of concept. Exudate Progress Monitor dots do not change appearance with/without exudate.

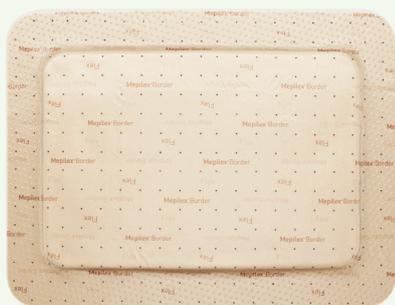

Exudate Progress Monitor is a grid of equidistant dots that can be used to track and record exudate.

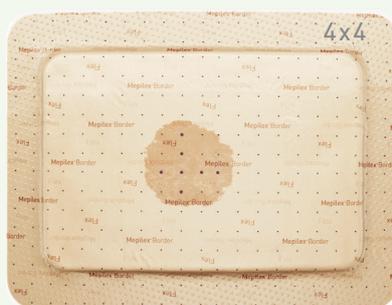

As exudate spreads, you can record a dot count\*\* that reflects fluid volume in the dressing.

\*\* All images are stylised for ease of visualisation of concept. Exudate Progress Monitor dots do not change appearance with/without exudate



Any staff can then refer to the recorded dot count\*\* to help monitor exudate trends.

### Mepilex Border Flex Assortment

Product Code	Size (cm)
595211	7.5 x 7.5
595311	10 x 10
595011	12.5 x 12.5
595411	15 x 15
595611	15 x 20

### Areas of use

Mepilex Border Flex is designed for a wide range of exuding chronic and acute wounds, such as diabetic foot ulcers, venous leg ulcers, pressure injuries\*, skin tears and traumatic wounds.

\* Where a repositioning or offloading protocol is used

### Frequency of change

Wear time of up to seven days<sup>1,2,3</sup>, depending on the condition of the wound or as per clinical judgement.



References: 1. ProDerm. Assessment of wearing properties of wound dressings on the knees. Report no. PD-535012. 22 August 2016. Data on file. 2. ProDerm. Assessment of wearing properties of wound dressings on the elbows. Report no. PD-535013. 22 August 2016. Data on file. 3. Registry study Bordered foam products (including Mepilex Border) GW2016. 4. Mölnlycke Health Care. Mepilex Border Flex – Absorption and dispersion tests on inclined plane, viscous test solution. Report no. PD-528871. Data on file. 5. Mölnlycke Health Care. Mepilex Border Flex - Fluid handling capacity. Report no. PD-527642. Data on file. Data on file. 6. Mölnlycke Health Care. Mepilex Border Flex - Bacteria encapsulation. Report no. PD-537072. Data on file.

  
**Mölnlycke<sup>®</sup>**

# How to apply Mepilex® Border Flex

How to apply  
Mepilex Border Flex  
15x15 cm on a leg ulcer.




1. Remove the middle release liner.



2. Apply to wound.



3. Remove second release liner.

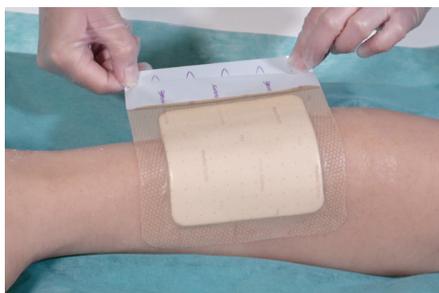

4. Remove third release liner.



5. Smooth down the borders.


How to apply  
Mepilex Border Flex  
10x10 cm on a foot ulcer.



1. Hold the larger section of the release liner and partially peel back



2. Position the exposed part of the dressing over the wound and apply

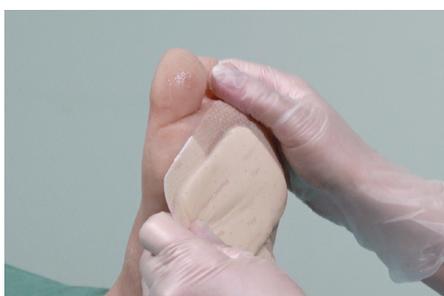

3. Continue to remove the larger section of the release liner and smooth down the borders



4. Remove the second release liner.



5. Smooth down the borders.

  
**Mölnlycke®**