



eFlow<sup>®</sup> rapid  
nebuliser system

The new eFlow<sup>®</sup> rapid  
with eBase Controller

**New!**  
**with Display.**

[www.eFlowrapid.info](http://www.eFlowrapid.info)

PARI Pharma  
Advancing Aerosol Therapies



# The new eFlow<sup>®</sup> *rapid* with eBase Controller



Compatible:



**eFlow<sup>®</sup> *rapid* nebuliser handset e.g. with:**

- Colistin
- Tobramycin
- DNase
- Hypertonic saline solution (MucoClear<sup>®</sup> 3% and MucoClear<sup>®</sup> 6%)

Compatible:



**Drug-specific nebuliser handset:**

- e.g. Altera<sup>®</sup> for Cayston<sup>®</sup> (aztreonam lysine)
- Other drug-specific nebuliser handsets used in multiple clinical trials


## Display Information

The new eFlow<sup>®</sup>rapid with eBase controller can do much more:




### **Battery charge level 50 %**

-> Have new batteries on hand or operate with the power adapter connected

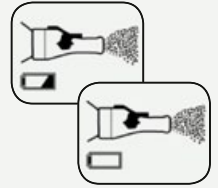


### **Cable connection to the control unit or on the nebuliser handset interrupted**

-> Check cable connections

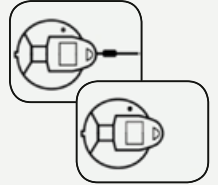

### **No medication added**

-> Add medication

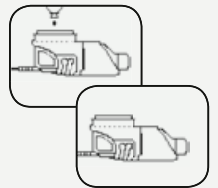

## Highest Standards of Hygiene for the Nebuliser Handset\*

- Can be disinfected in steam disinfectant
- Can be disinfected by boiling in distilled water
- Autoclavable (up to 121 °C)

\*Please refer to the Instructions for Use.

## Worldwide Mobility – Small, Light, Quiet

- Can be run on rechargeable or disposable batteries
- Mains adapter for Europe, US & UK
- Weight approx. 300 g, incl. batteries
- Quiet and discrete to use
- 2 complete nebuliser handsets included to change whilst travelling



**eFlow<sup>®</sup>rapid**  
nebuliser system

Order No. 178G1005

Replacement parts **eFlow<sup>®</sup>rapid** with eBase Controller  
(control unit blue)



**eFlow<sup>®</sup>rapid**  
**nebuliser handset 2.0**  
(incl. aerosol head)  
Order No. 678G8222



**eFlow<sup>®</sup>rapid aerosol head 2.0**  
Order No. 678B2620

Replacement parts **eFlow<sup>®</sup>rapid**  
(control unit turquoise)



**eFlow<sup>®</sup>rapid**  
**nebuliser handset**  
(incl. aerosol head)  
Order No. 178G8012



**eFlow<sup>®</sup>rapid aerosol head**  
Order No. 178B2603

**PARI**  
Export Service  
Centre

**+49 (0) 8151 279-220**

**Please give us a ring ...**

You can find PARI on the Internet:

E-Mail: [info@pari.de](mailto:info@pari.de)

[www.eflowrapid.info](http://www.eflowrapid.info)



**PARI Pharma GmbH**  
Moosstraße 3 · D-82319 Starnberg

**PARI Pharma**  
Advancing Aerosol Therapies

