


BD Insyte™ Autoguard™ BC Pro




Shielded IV Cannula with Blood Control Technology

Safety IV cannula designed to reduce blood exposure* and needlestick injury

No need to worry about stopping blood during insertion**

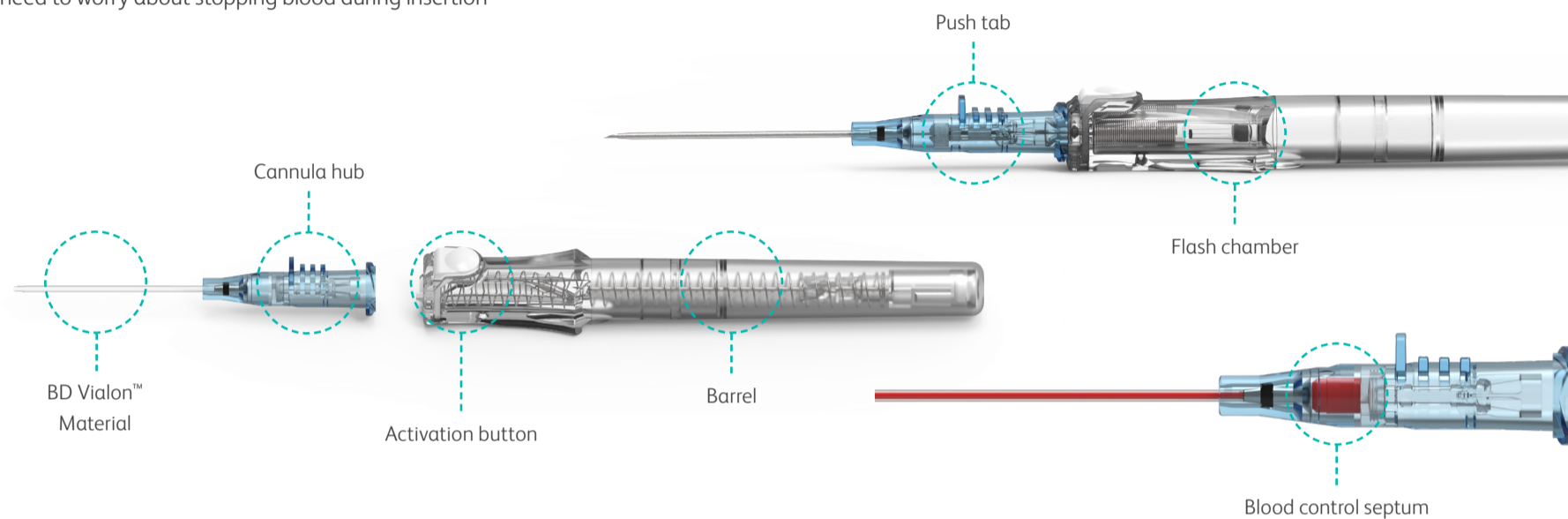

Points to practice

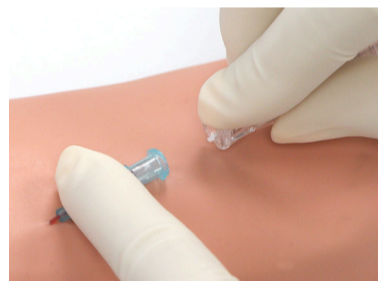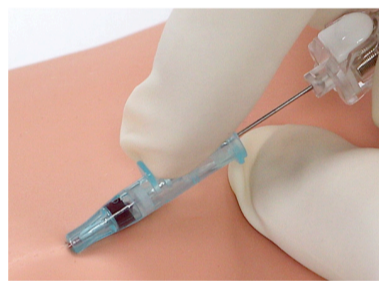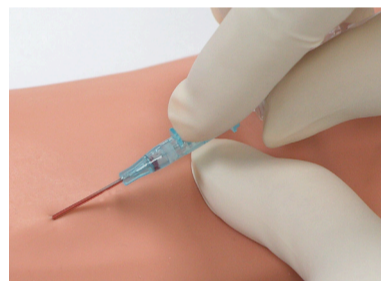

Preparation

- Holding the cannula hub, rotate the barrel 360°
- Make sure the push tab is facing upwards and the cannula is seated back in the notch

Insertion and blood return

- Stabilise the vessel and perform the venipuncture
- Look at the cannula for initial blood return (18–24 G), then in the flash chamber behind the white button
 - In 16 G, blood return will only be in the flash chamber

Advancement

- Once you see blood return, lower the cannula angle and advance the entire cannula and needle unit slightly to ensure the cannula tip and not just the needle tip is within the vessel
- While stabilising the vessel, push the cannula off the needle and into the vessel. Avoid pulling the needle out.

Needle retraction

- Release the tourniquet
- Gently stabilise the cannula hub
- Push the white button to retract the needle
- It is not necessary to occlude the vessel as blood flow will be restricted** until a secure luer connection is made opening the septum

Connection

- Firmly push and turn completely to make a secure connection. With movable spin collar connection, pull the spin collar back, make the connection and then secure the spin collar.
- Disconnection of the luer requires vessel compression to prevent blood leakage

Stabilisation

- Stabilise the cannula and apply a sterile dressing according to your policy

Tips for success

Insertion success

- Release the tip seal before insertion by rotating the barrel 360°
- Make sure the cannula is seated back in the notch with the push tab facing up
- Lower the initial insertion angle
- Stabilise and straighten the vessel

Getting blood return

- Look for initial return along the cannula with 18–24 G
- With 16 G, look behind the white button
- The septum keeps the blood contained in the coloured hub until a secure luer connection is made**

Threading with ease

- Rotate the barrel 360° before inserting the cannula
- Once blood return is seen, lower and advance the entire cannula and needle unit slightly to ensure the cannula tip is within the vessel
- Push the cannula forward off the needle
- Avoid pulling back on the needle
- Keep the vessel stabilised and straight

Retracting the needle

- Do not put any tension on the needle when pushing the button

* Compared to a non-blood-control cannula.

**For up to 10 seconds.

Consult product insert for complete instructions, warnings and cautions.

Becton Dickinson Pty Ltd, Australia. Toll free: 1800 656 100

Becton Dickinson Limited, New Zealand. Toll free: 0800 572 468

bd.com

BD, the BD Logo, Autoguard, Insyte and Vialon are trademarks of Becton, Dickinson and Company or its affiliates.
© 2022 BD. All rights reserved. BD-40035

