

ALLEVYN[◇] GENTLE BORDER

Smith+Nephew

ALLEVYN[◇]
GENTLE BORDER
Foam Dressing

Dressing location guide

One in three pressure injuries in hospitalised adult patients are related to medical devices¹



Device

Cervical collar (front)

Area at risk

Chin, jaw, clavicle, occiput



Device

Cervical collar (back)

Area at risk

Occiput



Device

Nasogastric tubes/
(hi flo) nasal cannulas

Area at risk

Nose, upper lip, cheek, ear



Device

(NIPPV) Non-invasive positive
pressure ventilation/ CPAP

Area at risk

Forehead, nose, cheek, chin



Device

Trach flange

Area at risk

Neck



Device

SCD (Sequential compression device)

Area at risk

Lateral anterior
ankle, achilles,
top of foot



Device

Gastric tube

Area at risk

Stomach



Device

Multi-podus boot

Area at risk

Top/bottom foot, heel, calf



In accordance with your protocol,
continue to lift and/or move the medical
device for regular skin inspections and
reposition for pressure relief¹



ALLEVYN GENTLE BORDER can be
cut when placed on intact skin², lifted
and repositioned without losing its
adherent properties³

ALLEVYN[◇] family



Device

Brace

Area at risk

Hand



Device

Multi-podus boot

Area at risk

Top/bottom foot, heel, calf



ALLEVYN[◇] GENTLE BORDER

Product number	Dressing size: border to border	Pad size
66800269	7.5cm x 7.5cm	5cm x 5cm
66800270	10cm x 10cm	7.5cm x 7.5cm
66800272	12.5cm x 12.5cm	10cm x 10cm
66800273	17.5cm x 17.5cm	15cm x 15cm
66800506	Heel 25cm x 25.2cm	20cm x 20.2cm
66800959	Multisite 17.1cm x 17.9cm	11.6cm x 12.3cm
66800898	Sacrum Small 17.2cm x 17.5cm	12.3cm x 8.3cm
66801031	Sacrum Large 21.6cm x 23cm	17cm x 12.3cm
66800833	GB Lite 5cm x 5cm	2.5cm x 2.5cm
66801041	GB Lite Multisite 8cm x 8.4cm	5.5cm x 5.9cm

Smith & Nephew Pty Ltd Australia
 T +61 2 9857 3999
 F +61 2 9857 3900

Smith & Nephew Ltd New Zealand
 T +64 9 820 2840
 F +64 9 820 2841

www.smith-nephew.com/australia
www.smith-nephew.com/new-zealand
[◇]Trademark of Smith & Nephew. All trademarks acknowledged. 24609-anz V1 SN14876 REV0 04/20

For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.