

 **smith&nephew**

PICO[◊]

Single Use
Negative Pressure
Wound Therapy System

ACTICOAT[◊] **FLEX**

Antimicrobial Barrier Dressing



Take Control

Your choice of post-operative dressings may help mitigate the risk of a surgical site infection or complication

Preventing and reducing the rate of surgical site infection is critical to improving outcomes for patients and reducing healthcare burden

The goals:

- Optimised outcomes for patients
- Reduced healthcare burden associated with shorter length of stay
- Reduced complications, reduced costs of treating surgical site infections (SSI)



Increased length of stay to treat an SSI¹



Attributable to SSI following hip or knee replacement²



Economic burden of SSI³

* Calculated based on data from Graves et al 2009

Smith & Nephew is committed to:

- Putting patients first
- Helping to reduce the risk of surgical site complications and infections
- Helping to reduce the risk of preventable readmissions



Prevention through innovation

PICO^o

Single Use Negative Pressure Wound Therapy System

The modes of action of NPWT improves the speed, strength and quality of incisional wound healing^{4,5} thus minimizing the failures of healing that lead to infection and/or dehiscence.



Reduce complications and readmissions

May help reduce complications and readmissions as part of a comprehensive clinical protocol.^{6,7}



Early Discharge

Allows earlier patient discharge which may result in cost savings to hospitals.⁸



Patient friendly

PICO allows patients to be comfortable, shower and perform everyday activities.⁹



Take control

ACTICOAT^o Flex

Antimicrobial Barrier Dressings for use on high risk and infected surgical wounds



Fast acting

Starts working within 30 minutes (*in vitro*).¹⁰⁻¹²



Kills 150+ pathogens including MRSA, VRE, CRE and fungi *in vitro*

Broad spectrum antimicrobial activity offers reliable protection.¹³⁻¹⁷



Compatible with NPWT

ACTICOAT Flex can be used in combination with NPWT for up to 3 days.

Ordering information

Product	Size (cm)	Code	Qty
PICO®	10 x 20	66801358	Kit/1
ACTICOAT® Flex 3	4 x 15	66801290	Box/5
PICO®	10 x 30	66801359	Kit/1
ACTICOAT® Flex 3	4 x 25	66801291	Box/5
PICO®	10 x 40	66801360	Kit/1
ACTICOAT® Flex 3	4 x 35	66801292	Box/5
PICO®	15 x 30	66801363	Kit/1
ACTICOAT® Flex 3	10 x 20	66800409	Box/12
ACTICOAT® Flex 7	2.5 x 60	66800545	Box/5




Dress, Press and Go

Dress




Add ACTICOAT Flex to incision.

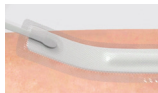

Partially remove backing from dressing and position dressing over incision. Ensure the port is not placed over the incision.



Remove remaining backing and smooth dressing edges to secure to periwound skin.

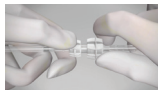

Connect dressing port tubing to pump tubing and twist to secure the connection.

Press



Push the orange button to start delivery of NPWT.



Seal dressing with the supplied adhesive retention strips around all edges.

Go



Change dressing as needed within the 7-day period (pump life) using the additional dressing provided, if needed. ACTICOAT Flex may be used as a wound contact layer in combination with NPWT for a period of up to 3 days.

References

- Jenks PJ, et al. Clinical and economic burden of surgical site infection (SSI) and predicted financial consequences of elimination of SSI from an English hospital. *Journal of Hospital Infection*; 86(1): 24-33.
- Peel TN, et al. Direct hospital cost determinants following hip and knee arthroplasty. *Arthritis Care & Research* 2015; 67(6): 782-790.
- Graves N, et al. Economic rationale for infection control in Australian hospitals. *Healthcare Infection* 2009;14: 81-88.
- Cornhill AH, et al. Incisional vacuum-assisted closure therapy. *J Orthop Trauma* 2006; 20: 705-709.
- Driffield K. ACTICOAT Flex 7 has antimicrobial activity in 30 minutes. Data on file; 2008: report 0810018.
- Yin HQ, et al. Comparative evaluation of the antimicrobial activity of ACTICOAT antimicrobial barrier dressing. *J Burn Care Rehab* 1999; 20: 195-200.
- Westlain (Sheriff) Report. Broad spectrum efficacy. Data on file; 1993: report 93/001.
- Wright JB, et al. Efficacy of topical silver against fungal burn wound pathogens. *Am J Infect Cont* 1999; 27(4): 344-350.
- Lumb H. Antimicrobial activity of the ACTICOAT range of dressings and Aquacel Ag™ against *P. aeruginosa* and MRSA over a 7 day period. Data on file; 2011: report 1102007.
- Wright JB, et al. Wound management in an era of increasing bacterial antibiotic resistance: a role for topical silver treatment. *Am J Infect Cont* 1998; 26(6): 572-577.
- Hope R, et al. The in-vitro antibacterial activity of nanocrystalline silver dressings against bacteria with NDW-1 carbapenemase. Poster presented at EWMA, Vienna, 2012.

Smith & Nephew Pty Ltd

Australia
T 61 2 9857 3999 F 61 2 9857 3900
www.smith-nephew.com/australia
Customer Service
T 13 13 60 F 1800 671 000

New Zealand
T 64 9 820 2840 F 64 9 820 2841
www.smith-nephew.com/new-zealand
Customer Service
T 0800 807 663 F 0800 263 222

Please consult product labels and inserts for any indications, contraindications, hazards, warnings, cautions and instructions for use
©2016 Smith & Nephew, Inc. All rights reserved.

[®]Trademark of Smith & Nephew
SN12488 (06/16)